



## SYMMETRISLEEP WASHING INSTRUCTIONS

Description	Sizes	Cm.	Washing Instructions
VELCRO SHEET c/w FASTENINGS	COT SIZE	100 x 75	40 Degree wash and the Velcro to be 'closed' (attached to the sheet so the loops aren't exposed to the washing action).
	SINGLE	150 x 100	
	EXTENDED TIES	150 x 100	
POSITIONING BRACKETS c/w Velcro	SMALL(PAIR)	10x13x8	Wipe clean with a soft cloth moistened with water and neutral detergent DO NOT use Bleach or biological cleaners.
	MEDIUM(PAIR)	16x13x13	
	LARGE(EACH)	15x25x25	
	EX- LARGE(EACH)	30x25x25	
POSITIONING BRACKET PADS	SMALL(PAIR)	17 x 15	Wipe clean with a soft cloth moistened with water and neutral detergent DO NOT use Bleach or biological cleaners.
	MEDIUM(PAIR)	20 x 25	
	LARGE(EACH)	25 x 28	
	EX- LARGE(EACH)	25 x 35	
SOFT KNEE BRACE	SMALL		Fleece Inserts - Can be washed up to 71 Degrees, Spin Dry, Autoclave, Lightly brush to restore pile. DO NOT Fold, Iron or Tumble Dry.
	MEDIUM		
	LARGE		
SIDE LYING LEG SUPPORT	SMALL		Wipe clean with a soft cloth moistened with water and neutral detergent DO NOT use Bleach or biological cleaners.
	MEDIUM		
	LARGE		
AIRMANTLE	SIZE 8 (LARGE)	190x100x20	Wash inner and outer as Elan cover below.
LOWZONE OVERMANTLE (ALL WITH ELAN COVERS)	SIZE 1 (COT)	123x63x15	Elan Cover - As below
	SIZE 2 (SMALL)	145x90x15	
	SIZE 3 (MEDIUM)	145x90x25	
	SIZE 4 (LARGE)	190x100x25	
COOLISCH	SIZE 1 (SMALL)	38 x 51	Elan Cover - As below
	SIZE 2 (LARGE)	76 x 51	
AIRSHEET	SIZE 1 (SMALL)	125 x 60	Wash with mild detergents at 40 degrees C. DO NOT Tumble Dry.
	SIZE 2 (LARGE)	145 x 90	
ELASTICATED OVERSHEET	FITTED COT	163x94	Wash at 40 degrees, DO NOT Iron, DO NOT Tumble dry and DO NOT Dry Clean.
	FITTED SINGLE	220x140	
	FITTED DOUBLE	234x168	
LOWZONE CRUMB FILLED CUSHIONS c/w IVORY BREATHABLE COVER	A-SQUARE	30x30	Terry Towelling Cover - Wash at 40 degrees, DO NOT iron, DO NOT Tumble Dry and DO NOT Dry Clean. Elan cover - Wipe clean with soft cloth moistened with water and neutral detergent DO NOT USE Bleach or Biological Cleaners. Machine Wash or Dry Clean up to 60 degrees Celsius (some surface wrinkling may occur. This has no adverse effect on the fabrics properties). Hang or Tumble Dry (on a cool setting).
	C-MEDIUM OBLONG	30x45	
	D-LARGE OBLONG	45x75	
	E-HORSESHOE	35x80	
	F-SMALL ROLL	8x30	
	G-MEDIUM ROLL	13x40	
	SIDE LYING BOLSTER		
	H-SQUARE	30x30	
POLYSTYRENE BEAD CUSHIONS c/w BLUE BREATHABLE COVER	J-MEDIUM OBLONG	30x45	Terry towelling cover - Wash at 40 degrees, DO NOT Iron, DO NOT Tumble Dry and DO NOT Dry Clean. Elan Cover - As above
	K-LARGE OBLONG	45x75	
	L-HORSESHOE	35x80	
	M-MEDIUM ROLL	13x40	
LOWZONE REPLACEMENT MATTRESS	SINGLE ADULT SPEC	200x90x10	Wipe clean with a soft cloth moistened with water and neutral detergent DO NOT use Bleach to biological cleaners.